

## LIFECLEAN DISINFECTANT USER INSTRUCTIONS

Premixed solution for cleaning and disinfection of surfaces with proven efficacy against bacteria, viruses, fungi and spores.

### Effective against

Vegetative bacteria, viruses, fungi, mycobacteria (e.g. TB) within 1 minute and endospores, for example *Clostridium difficile* and *Bacillus subtilis* within 2 minutes.

### Ready to be used

LifeClean is a premixed solution and is ready to be used straight from the bottle. This ensures that the solution has the right concentration and is easy to use.

### Content

ClO<sub>2</sub>  
pH

Nominal 200 ppm  
Acidic solution

### Method of surface disinfection

in accordance with the local applicable procedure:

#### Surfaces

- 1 Remove any visible contamination and previously applied chemicals.
- 2 Apply generous amount of LifeClean solution on a clean cloth and wipe the intended surface. Duration of action within 2 minutes.
- 3 After disinfection, wipe off the surface if there's uncertainty about the material compatibility with the disinfectant solution. Always follow the local routine guidelines.
- 4 Rinse off the surfaces with water every now and then.
- 5 For personal protection follow the local guidelines.
- 6 Close opened bottle and mark with opening day. An opened bottle is valid for 4 weeks from the opening day.
- 7 Remaining liquid can be poured in the drain, then put the bottle for recycling. Acc. to applicable guidelines, usage of LifeClean Disinfectant is not considered dangerous for the health or the environment.

#### Cleaning Tools

- 1 Clean the tool in advance, remove visible contamination.
- 2 Put the tool in a container and cover it with LifeClean solution.
- 3 Leave the solution for 2 minutes.
- 4 Clean, rinse and dry thoroughly.

**For information regarding personal protection, stability, and material compatibility please look at the actual safety data sheet.**