



LIFE CLEAN



Mini guide

Hospitals, Dentists & Clinics

- **Surface disinfectant**
- **Ready to use**
- **2 minute contact time**

Can be used to disinfect for example:

- Bedrails
- Call bell
- Telephones
- TV remote
- IV pump / pole
- Toilet / commode chair
- Overbed table
- Light switches
- Doorknobs
- Respiratory and other bedside equipment
- Computer keyboard
- Chairs
- Screen Curtain
- Semi-critical instruments



Norovirus | *Clostridium difficile* | MRSA | ESBL

For complete list see [Microbiological Efficacy Summary](#).



Watch the video



LIFECLEAN

HEALTHCARE

www.lifeclean.se

LIFE CLEAN

HEALTHCARE

How to apply



1. Open the bottle and write the opening date on the bottle.



2. Allow LifeClean to operate for 2 minutes. Rinse / dry if necessary, always follow the local guidelines. When in contact with metal surfaces rinse or wipe after contact time.



Note! Dirty surfaces should always be cleaned before application of LifeClean Disinfectant



Tip! Place the tool in a container with LifeClean. If the tool can not be soaked with LifeClean, apply LifeClean directly to the tool.

For information regarding personal protection, stability, and material compatibility please look at the actual safety data sheet.

Opened bottle is valid for 4 weeks and an unopened bottle is valid for 9 months.

Application techniques



LifeClean bottle & Microfiber cloth



LifeClean bottle with trigger spray



Electrostatic spray (Hand-held/Backpack)

