

LIFECLEAN DISINFECTANT USER INSTRUCTIONS

Premixed solution for cleaning and disinfection of surfaces with proven efficacy against bacteria, viruses, fungi and spores.

Effective against

Vegetative bacteria, viruses, fungi, mycobacteria (e.g. TB) within 1 minute and endospores, for example *Clostridium difficile* and *Bacillus subtilis* within 2 minutes.

Ready to be used

LifeClean is a premixed solution and is ready to be used straight from the bottle. This ensures that the solution has the right concentration and is easy to use.

Content

ClO₂
pH

Nominal 200 ppm
Acidic solution

Method of surface disinfection

in accordance with the local applicable procedure:

Surfaces		Cleaning Tools	
1	Remove any visible contamination and previously applied chemicals.	1	Clean the tool in advance, remove visible contamination.
2	Apply generous amount of LifeClean solution on a clean cloth and wipe the intended surface. Duration of action within 2 minutes.	2	Put the tool in a container and cover it with LifeClean solution.
3	After disinfection, wipe off the surface if there's uncertainty about the material compatibility with the disinfectant solution. Always follow the local routine guidelines.	3	Leave the solution for 2 minutes.
4	Rinse off the surfaces with water every now and then.	4	Clean, rinse and dry thoroughly.
5	For personal protection follow the local guidelines.	<p>For information regarding personal protection, stability, and material compatibility please look at the actual safety data sheet.</p>	
6	Close opened bottle and mark with opening day. An opened bottle is valid for 4 weeks from the opening day.		
7	Remaining liquid can be poured in the drain, then put the bottle for recycling. Acc. to applicable guidelines, usage of LifeClean Disinfectant is not considered dangerous for the health or the environment.		